

Type MA Biomedical Chip NTC Thermistors

Thermometrics Biomedical Chip Thermistor Assemblies are designed for use in applications involving both intermittent and continuous patient temperature monitoring. Repeatability and fast



response are essential, not only for the intermittent temperature requirements associated with oral and rectal fever measurements, but also for the continuous monitoring often necessary during induced-hypothermia and general anesthesia, or when employed in the care of infants and premature babies. Intensive care units, along with recovery rooms, have also adopted patient temperature as part of their vital sign monitoring procedures. Temperature monitoring for skin surface, tympanic, esophageal, foley catheters, and biofeedback applications has also improved due to the high stability and tight interchangeable tolerances designed into each Thermometrics' Biomedical Sensor Assembly.

Within our Type MA line of NTC Thermistors, we offer a complete line of standard sub-assemblies. Continuous research and development efforts within Thermometrics have resulted not only in these field-proven designs, but in the development of our Unitherm Thermochip Thermistor, designed exclusively for biomedical applications in the range of 32°F to 122°F (0°C to 50°C).

Although low in cost, these highly stable, precision thermochips provide the reliability, tight interchangeable tolerances, geometries, and fast response times that are often required in medical applications.

Thermometrics employs experienced applications engineers who welcome inquiries, whether for assistance, or to help with your design requirements concerning new and existing applications.

Applications

- Intermittent Patient Temperature Monitoring For oral and rectal fever measurements
- Continuous Patient Temperature Monitoring During general anesthesia, induced hypothermia, and premature infant care
- Vital Sign Monitoring For Intensive Care Units (ICU) and recovery rooms
- Temperature Monitoring For skin surface, tympanic, esophageal, foley catheters, and biofeedback

Amphenol Advanced Sensors

Type MA Specifications

MA100



Designed for use in catheter assemblies, these sensors are available with nominal resistance values of 2252, 3000, 5000, and 10,000 Ω at 77°F (25°C).

Close monitoring of manufacturing processes allow Thermometrics to maintain tight interchangeability tolerances with volume production.

Typical design parameters are represented in Assembly Type table.

- 0.030 in (0.76 mm) (10 KΩ only) and 0.050 in (1.27mm) diameters available with kapton sleeve only.
- 0.070 in (1.78 mm) diameter normally supplied with molded plastic tip.

MA200



Intermittent body temperature measurements are common practice in all phases of patient care. This assembly is ideally suited for the disposable cover oral and rectal fever thermometers in use today. It features rugged construction with tip sensitive shaft assemblies exhibiting resistance values of 2252, 3000, 5000, and 10,000 Ω at 77°F (25°C). Refer to Assembly Type table for typical design parameters.

MA300



Routine continuous patient temperature monitoring is now feasible by using the convenience of the patient's skin site as an indicator of body temperature. The stainless steel housing used is suitable for both reusable and disposable applications, while maintaining maximum patient comfort. Nominal resistance values of 2252, 3000, 5000, and 10,000 Ω at 77°F (25°C) are available. Refer to Assembly Type table for typical design parameters.

Assembly Type

Assem- bly Type	MA100	MA200	MA300
Standard Diameters (D)	0.030 in (0.762 mm) 0.050 in (1.27 mm) 0.070 in (1.78 mm) 0.080 in (2.03 mm)	0.156 in (3.96 mm)	0.375 in (9.52 mm)
Body Length (L)	3/8 in (9.52 mm)	3.75 in (95.25 mm)	N/A
Standard Lead Length (A)	24 in (610mm) GG: 36 in (914mm)	2 in (51 mm)	24 in (610mm)
Tolerance	See Tolerance Code and Tem- perature Table	See Tolerance Code and Tem- perature Table	See Tolerance Code and Tem- perature Table
Wire Gauge	28, 30, 32, 38, AWG	30 AWG	30 AWG
Standard Wire Insulation	*Heavy Isomid *Medical Grade PVC *Polyurethane with Nylon Overcoat	PTFE	Medical Grade PVC PTFE
Body Material	Molded Plastic or Kapton Sleeve	Lexan Shaft Aluminum Tip	Stainless Steel
Nominal R Values @ 77°F (25°C)		2252 Ω, 3000 Ω, 5000 Ω, 10,000 Ω (for all three types)	

Thermal Response Time (63% Response)

Series	Still Air	Still Water*
MA100 Catherter Assembly	15 seconds	2.0 seconds
MA200 Oral-Rectal Assembly	35 seconds	0.6 seconds
MA300 Skin Surface Assembly	45 seconds	2.0 seconds
10	11 1 16 7705 (05)	20) i i 440E (500) i

Response time provided is for assembly plunged from 77°F (25°C) air to 41°F (5°C) water.

Type MA Specifications (Cont.)

Tolerance Code and Temperature Range

	То	lerance Code	9
Temperature Range °F (°C)	A ±°F (±°C)	B ±°F (±°C)	C ±°F (±°C)
32 to 68 (0 to 20)	0.27 (0.15)	0.36 (0.2)	0.45 (0.25)
68 to 95 (20 to 35)	0.18 (0.1)	0.27 (0.15)	0.36 (0.2)
95 to 102 (35 to 39)	0.09 (0.05)	0.18 (0.1)	0.27 (0.15)
102 to 107.60 (39 to 42)	0.14 (0.075	0.27 (0.15)	0.36 (0.2)
107.60 to 113 (42 to 45)	0.18 (0.1)	0.27 (0.15)	0.36 (0.2)
113 to 122 (45 to 50)	0.27 (0.15)	0.36 (0.2)	0.45 (0.25)

Data

Biomedical Chip Thermistors and sub-assemblies are designed to be interchangeable over a 32°F to 122°F (0°C to 50°C) range. Best overall stability is maintained when exposure and storage temperatures remain below 158°F (70°C).

Resistance vs Temperature

Temperature	, ,	0 10		1010	10000
°F (°C)	2252 Ω	3 κ Ω	5 κΩ	10 κΩ	103(Y) Ω
32.00 (0)	7373.00	9821.93	16369.9	32739.8	29565.8
33.80 (1)	7005.80	9332.76	15554.6	31109.2	28224.8
35.60 (2)	6659.04	8870.84	14784.7	29569.5	26951.9
37.40 (3)	6331.49	8434.49	14057.5	28115.0	25743.3
39.20 (4)	6021.97	8022.17	13370.3	26740.6	24595.5
41.00 (5)	5729.40	7632.41	12720.7	25441.4	23505.0
42.80 (6)	5452.74	7263.87	12106.4	24212.9	22468.7
44.60 (7)	5191.06	6915.27	11525.4	23050.9	21483.8
46.40 (8)	4943.46	6585.43	10975.7	21951.4	20547.2
48.20 (9)	4709.11	6273.23	10455.4	20910.8	19656.6
50.00 (10)	4487.22	5977.64	9962.74	19925.5	18809.3
51.80 (11)	4277.07	5697.69	9496.15	18992.3	18003.1
53.60 (12)	4077.97	5432.47	9054.11	18108.2	17235.7
55.40 (13)	3889.29	5181.11	8635.19	17270.4	16505.1
57.20 (14)	3710.42	4942.83	8238.05	16476.1	15809.4
59.00 (15)	3540.80	4716.88	7861.46	15722.9	15146.7
60.80 (16)	3379.91	4502.54	7504.24	15008.5	14515.3
62.60 (17)	3227.24	4299.17	7165.28	14330.6	13913.6
64.40 (18)	3082.34	4106.14	6843.57	13687.1	13340.0
66.20 (19)	2944.77	3922.88	6538.13	13076.3	12793.0
68.00 (20)	2814.12	3748.83	6248.05	12496.1	12271.4
69.80 (21)	2690.01	3583.49	5972.48	11945.0	11773.8
71.60 (22)	2572.07	3426.38	5710.63	11421.3	11299.0
73.40 (23)	2459.96	3277.03	5461.72	10923.4	10845.8
75.20 (24)	2353.37	3135.04	5225.07	10450.1	10413.1
77.00 (25)	2252.00	3000.00	5000.00	10000.0	10000.0
78.80 (26)	2155.56	2871.53	4785.88	9571.77	9605.42
80.60 (27)	2063.79	2749.28	4582.13	9164.26	9228.45
82.40 (28)	1976.44	2632.91	4388.19	8776.38	8868.24
84.20 (29)	1893.27	2522.12	4203.54	8407.07	8523.96
86.00 (30)	1814.07	2416.61	4027.68	8055.35	8194.82
87.80 (31)	1738.61	2316.09	3860.15	7720.30	7880.09
89.60 (32)	1666.71	2220.31	3700.51	7401.03	7579.07
91.40 (33)	1598.18	2129.02	3548.36	7096.72	7291.10
93.20 (34)	1532.85	2041.98	3403.30	6806.60	7015.55
95.00 (35)	1470.54	1958.98	3264.97	6529.94	6751.83
96.80 (36)	1411.11	1879.81	3133.02	6266.03	6499.37
98.60 (37)	1354.41	1804.27	3007.12	6014.23	6257.64
100.40 (38)	1300.29	1732.18	2886.96	5773.93	6026.13
102.20 (39)	1248.63	1663.36	2772.27	5544.53	5804.36
104.00 (40)	1199.30	1597.65	2662.75	5325.50	5591.88
105.80 (41)	1152.19	1534.89	2558.15	5116.30	5388.26
107.60 (42)	1107.19	1474.94	2458.23	4916.46	5193.09
109.40 (43)	1064.18	1417.65	2362.75	4725.49	5005.96
111.20 (44)	1023.08	1362.89	2271.49	4542.98	4826.53
113.00 (45)	983.78	1310.55	2184.24	4368.49	4654.43
114.80 (46)	946.21	1260.49	2100.82	4201.64	4489.33
116.60 (47)	910.27	1212.62	2021.03	4042.05	4330.91
118.40 (48)	875.89	1166.81	1944.69	3889.38	4178.88
120.20 (49)	842,99	1122.99	1871.65	3743.29	4032.94
122.00 (50)	811.50	1081.04	1801.73	3603.46	3892.83
· · · /				-	

Type MA Specifications (Cont.)

Ordering Information

The code number to be ordered may be specified as follows:

MA					
1	Code	Standard	Assemblies		
	100FA	Series 100	0, 0.070 in (1.7	8 mm) diameter molded plastic tip,	
		30 gauge	PVC insulated	ribbon cable.	
	100FD	Series 100	0, 0.070 in (1.7	8 mm) diameter molded plastic tip,	
		32 gauge	bifilar heavy is	omid insulation.	
	100DD	Series 100	0, 0.050 in (1.2	7 mm) diameter kapton sleeve with	
		32 gauge	bifilar heavy is	omid insulation.	
	100BF	Series 100	Series 100, 0.030 in (0.768 mm) diameter kapton sleeve with		
		38 gauge	bifilar heavy is	omid insulation.	
	200LC	Series 200, 0.156 in (3.96 mm) diameter aluminum tip,			
		30 gauge	PTFE leads.		
	3001A	Series 300	0, 0.375 in (9.5	2 mm) diameter stainless steel cup,	
	000TD	30 gauge	PVC insulated	ribbon cable.	
	3001B	Series 300, 0.375 in (9.52 mm) diameter stainless steel cup,			
	10000	30 gauge	PIFE Insulate	d ribbon cable.	
	TUUGG	Series 100, 0.080 in (2.03 mm) diameter molded plastic tip, 28 gauge kynar insulated twisted pair.			
		Code			
		202	3000 0		
		502	5000 \$2		
		103			
		103 (Y)	10,000 0		
			Code	Tolerance	
			X	Refer to tolerance code and temperature	
			Ĩ	range table	
V	↓				
MA		·	Typ	ical Model Number	

*Refer to resistance vs temperature table on previous page.

Amphenol Advanced Sensors

www.amphenol-sensors.com

© 2020 Amphenol Corporation. All Rights Reserved. Specifications are subject to change without notice. Other company names and product names used in this document are the registered trademarks or trademarks of their respective owners.